Hawe Implant Paste for Cleaning and Polishing

English
Hawe Paste for Implant Cleaning and Polishing
Product information
The special paste for a professional, efficient and gentle maintenance of implants. Please proceed with following recommended procedure: <ol style="list-style-type: none">Use a metal-free instrument to eliminate deposits (as for example our Implant Deplaquers, items 1350 (universal) or 1360 (orofacial)); Paste auf paste on a cup or a brush and treat the implant neck for ca. 20 seconds. <p>Please do not exceed a speed of 2000 rpms!</p>

All KerrHawe’s cups and/or brushes can be recommended for an implant hygiene session with this special paste. The best performance can be reached using following products: Pro-Cup soft (latex-free, item 990/30 for latch, 1810/30 for screw), screw-type soft point (item 1805/30), screw-type spiral soft rubber (item 1807/30), screw-type brush small (item 0220), screw-type brush small point (item 0240). Please ask for a product catalogue for further information about KerrHawe’s cups and brushes product portfolio.
RDA: 9,8 REA: 4,5

Order number:
Art. No. 3500 Tube of 45g, without fluoride, mint flavour
Order number:
Art. No. 3500 Tube of 45g, without fluoride, mint flavour

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		
Manufactured by KerrHawe SA	Kerr Australia Pty. Ltd.	Sybron Dental Specialities Japan, Inc.
Via Strece 4, 6934 Bioggio	Unit 11, 112-118 Talavera Road	29-24 Honkomagone 2-chome
Switzerland	North Ryde 2113	Bunkyo-ku, Tokyo
www.KerrHawe.com	New South Wales, Australia	113-0021 Japan
00800 41 05 05 05	61-29-870-7500	0120-18-3126

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