
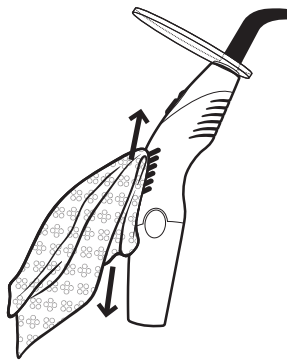



Demi

LED Light Curing System

Product Care Tips

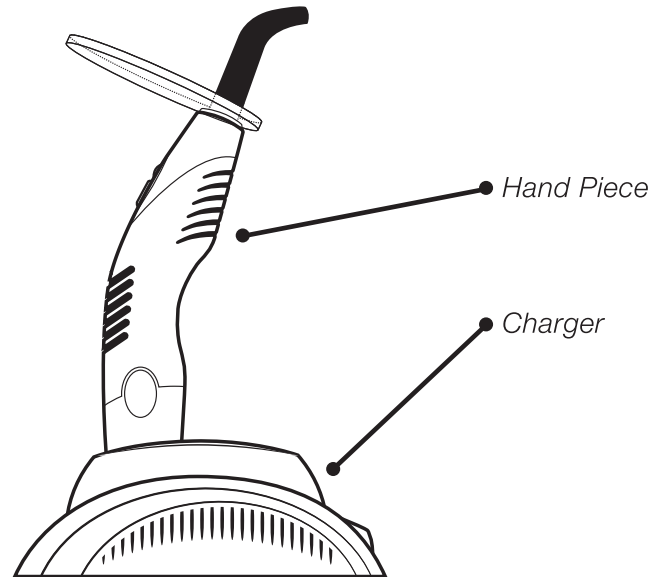


• **Do** wipe handpiece.

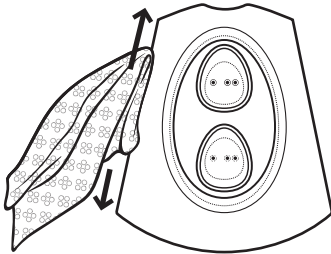


• **Do** wipe charger base.


- **Do** let handpiece air dry completely before placing into charger base.
- **Do** use Caviwipes™ or any disinfectants containing less than 20% alcohol.
- **Do** wipe handpiece separately from charger base.

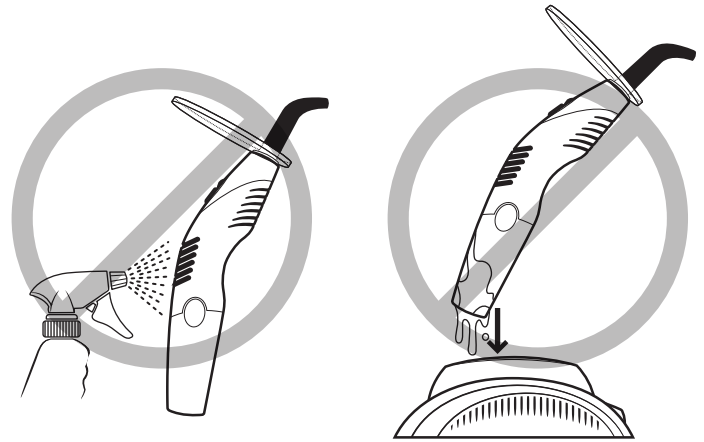


• **Do not** spray handpiece.

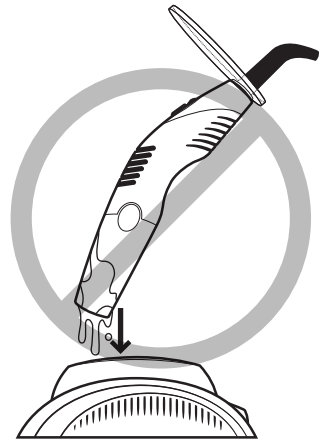

• **Do not** place wet handpiece in charger.

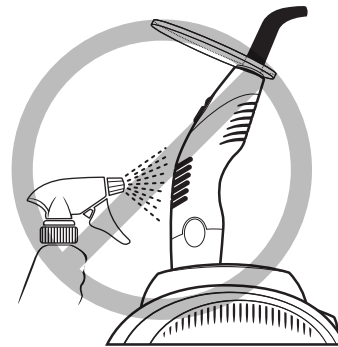

• **Do not** spray handpiece while in charger base.

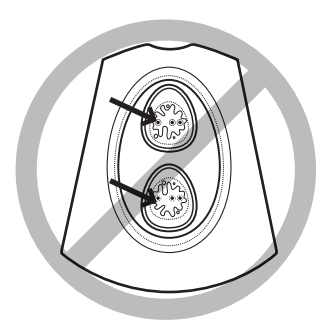

• **Do not** allow sockets in charger to get wet.

- **Do not** use denatured alcohol, Lysol®, phenol, ammonia, or iodine solutions.

FOR COMPLETE INSTRUCTIONS ON HOW TO MAINTAIN YOUR DEMI, PLEASE REFER TO THE DIRECTIONS FOR USE.